

# Operation Manual for Control Panel XMT-308

## Warm & Cozy Blanket Warmer

**1. Pls refer to "User's Manual" first. All of Warm & Cozy blanket warmer include "User's Manual" at the carton box.**

### **2. The temperature control**

1) The factory set the temperature at 130 F (Fahrenheit) +- 10%

After turn-on the power, pls wait about 30 minutes for it to fully heat up.

The temperature will be moved from 115 F to 145 F.

2) For re-set the temperature

a) PRESS & HOLD **the UP or DOWN arrow button** for about 2-3 seconds.

Then select the desired temperature setting by pushing the up or down arrow button to increase or to decrease the temperature.

b) Once a desired temperature has been selected, push **SET button** to save the temperature.

### **3. Re-Set the initial value (to make the factory re-set value)**

1) Pls refer to attached **XMT-308 Temperature Control Instruction Manual**

(at the 2<sup>nd</sup> page, pls refer to "First Menu" for ex-factory value.)

2) PRESS the SET button for 2-3 seconds.

Window display **INDEX SYMBOL** (ex, AL, SC, P, I, d, t, hy, At, Loc)

and **INDEX VALUE** (ex, 200, 0, 15.0, 240, 30, 10, 1.0, OFF, 0).

- 3) Re-Set the initial value
- a) Press **the SET button** for 2-3 seconds.
  - b) INDEX symbol "**AL**" will be display (High Limit Alarm)
    - ⇒ By using UP or Down button to make INDEX value with "**200**" and Press "SET" button one more time to save.
    - ⇒ Automatically the display will be next Index Symbol of "**SC**".
  - c) INDEX symbol "**SC**" → INDEX value "**0**"
  - d) INDEX symbol "**P**" → INDEX value "**15.0**"
  - e) INDEX symbol "**I**" → INDEX value "**240**"
  - f) INDEX symbol "**d**" → INDEX Value "**30**"
  - g) INDEX symbol "**t**" → INDEX value "**10**"
  - h) INDEX symbol "**HY**" → INDEX value "**1.0**"
  - i) INDEX symbol "**At**" → INDEX value "**OFF**"
  - j) INDEX Symbol "**Loc**" → INDEX value "**0**"
  - k) After 2-3 seconds, the current temperature will be displayed.

If you have any question, pls contact Point A Tech at any time.

- Point A Technologies, Inc.

20957 Currier Road, Unit C

Walnut, CA 91789

Tel : 909-869-8949 Fax; 909-869-8959

Attn : Greg H. Kim [greghk@pointatech.com](mailto:greghk@pointatech.com) [www.pointatech.com](http://www.pointatech.com)

Thks & B/Rdgs.

Point A Tech.